
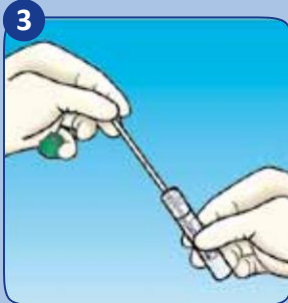
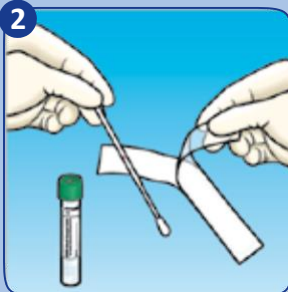


**HOW TO USE
BD™ ESwab
and Universal
Viral Transport**

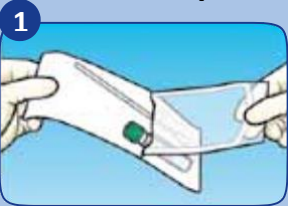

Microbiology Swabs

COLLECTION AND TRANSPORT GUIDE

BD ESwab Regular Collection Kit

White cap w/Liquid Amies and a regular flocked swab

- Intended uses:** Anaerobic/Aerobic Cultures
Fungal Cultures
AFB Cultures
Group B Strep PCR
MRSA Nasal PCR
Rapid Strep A
Rotavirus



BD ESwab Flexible Minitip Collection Kit

Blue cap w/Liquid Amies and flexible minitip flocked swab

- Intended uses:** Nasopharyngeal
Urethral
Pertussis/Parapertussis PCR

(When inserting, ensure that red line is below lip of tube prior to breaking)



BD Nasopharyngeal Swab / UVT Media Collection Kit

Red cap with flexible mini-tip flocked swab

Intended uses:

- Viral Culture
Rapid Flu A/B
Rapid RSV
Flu A/B PCR (incl.H1N1)
Viral Respiratory Panel PCR
Atypical pneumonia PCR
(Mycoplasma and Chlamydomphila)
Legionella PCR



Swab Collection Reminders:

- Have patient blow their nose (if nasal)
- Do not use lubricant (may use saline)
- Rotate 2-3 times
- Hold in place 5 seconds



BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2009 BD

